
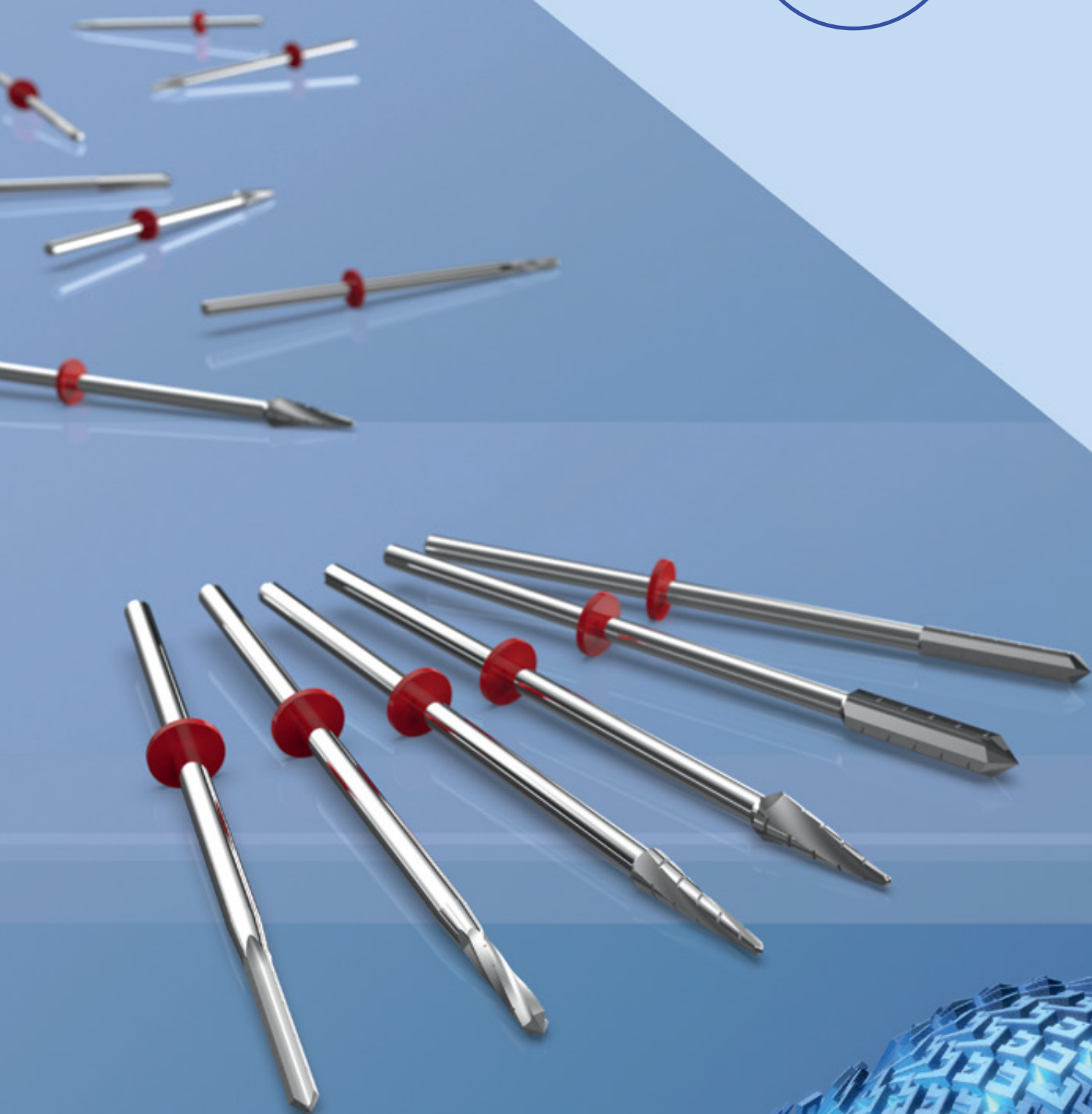
SURGICAL
TECHNIQUE GUIDE

INCLUDING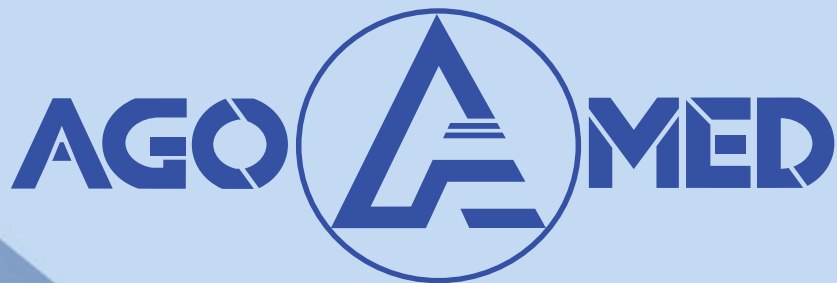

DPR SYSTEM

MINIMAL INVASIVE FOOT SURGERY

Made in Germany

EN Working with AGOMED means working with a company committed to excellence. Our products are German engineered, biocompatible, and cutting-edge. Our team members collectively bring decades of medical technology experience to the table.

We're reliable. We're flexible.
We're inventive.

AGOMED collaborates with distributors, hospitals, and doctors worldwide to create and develop exacting, state-of-the-art solutions. We work closely with specialists to insure that our implants for trauma and extremities improve the patient's quality of life.

Patient safety is always our number one priority.

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QUALITY IS PART OF
OUR DNA

DIN EN ISO
13485

AGOMED MEANS QUALITY.
WE EXCEED ISO 13485 STANDARDS
AND HAVE CE CERTIFICATION.

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CE
CERTIFICATE

INDEX

FEATURES	2
SURGICAL TECHNIQUE	3
CURETTES	7
INSTRUMENTS AND TOOLS	8

SET S1469

DPR SYSTEM MINIMAL INVASIVE FOOT SURGERY

FEATURES

The minimal-invasive surgery reveals new treatments perspectives for many foot pathologies. Agomed, specialist and precursor, wishes to succeed in this avant-garde technique. This complete minimal invasive solution was designed to answer demands of such surgery: precision, efficacy and rapidity.

This system includes:

- 6 burrs dedicated to each indication.
- 4 specific curettes to ease the surgical procedure.
- A soft tissue protector to bring more safety.

TECHNICAL FEATURES

- Single use: Sterile packaging.
- Safety: Protection by silicone ring.
- Identification: Color coded.
- Superior cut quality.
- Specific and unique designs.
- Standard attachment diam. 2.0 mm.

SURGICAL TECHNIQUE

STEP 1 - STRAIGHT BURRS

1.1. Straight Burr dia 2.0 - L. 15 mm Ref. 1201155

Specificity

Cutting & Extraction design

Possible uses

- 1st metatarsal distal osteotomies (mono or bi-cortical).
Eg: Hallux valgus treatment.
- 1st metatarsal proximal / basal osteotomies (mono or bi-cortical).
Eg: Severe Hallux valgus treatment.
- 1st phalanx proximal osteotomy / Akin osteotomy.
Eg: Hallux valgus treatment.
- Lesser rays osteotomies.
Eg: Metatarsalgia treatment.

1.2. Straight Burr dia 1.9 - L. 10 mm Ref. 1201157

Specificity

Cutting & Extraction design

Possible uses

- Lesser rays joints fusion.
Eg: Hammer toe treatment.
- Lesser rays phalangeal osteotomies.
Eg: Claw toe treatment.
- Small exostosis reaming.
Eg: 5th ray bunionectomy.

STEP 2 - CONICAL BURRS

2.1. Conical Burr dia 3.1/1.0 – L. 12 mm Ref. 1201156


Specificity

Conical Cutting & Extraction design

Possible uses

- 1st metatarsal proximal osteotomy.
Eg: Hallux valgus with DMMA deformity treatment.
- 1st phalanx proximal osteotomy / Akin osteotomy.
Eg: Hallux valgus treatment.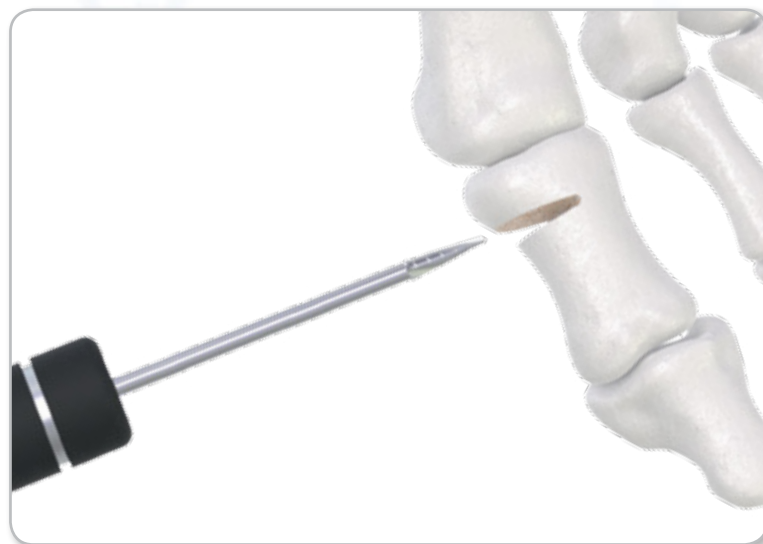

2.2. Conical Burr dia 4.1/1.0 – L. 12 mm Ref. 1201158

Specificity

Conical Cutting & Extraction design

Possible uses

- 1st metatarsal distal mono cortical osteotomies.
Eg: Hallux valgus with severe DMMA deformity treatment.
- 1st metatarsal proximal osteotomy / Akin osteotomy.
Eg: Hallux valgus treatment.

STEP 3 - CYLINDRICAL BURRS

3.1. Cylindrical Burr dia 3.1 – L. 15 mm Ref. 1201160

Specificity

Reaming design

Possible uses

- Bunionectomy (medial or dorsal) of the first ray.
Eg: Hallux valgus treatment and Hallux Rigidus treatment.
- Mid & hind foot joint fusion.
Eg: Mid & Hind foot arthritis treatment.
- Hind foot exostosis reaming.
Eg: Haglund Syndrom treatment.

3.2. Cylindrical Burr dia 4.1 – L. 15mm Ref. 1201159

Specificity

Reaming design

Possible uses

- Bunionectomy (medial or dorsal) of the first ray, large bunion.
Eg: Hallux valgus and Hallux Rigidus treatment.
- Mid & hind foot joint fusion: large joint.
Eg: Mid & Hind foot arthritis treatment.
- Hind foot exostosis reaming.
Eg: Haglund Syndrom treatment.


STEP 4 - BURRS' SPIKES


Straight burrs
for cutting & extraction

Conical burrs
for reaming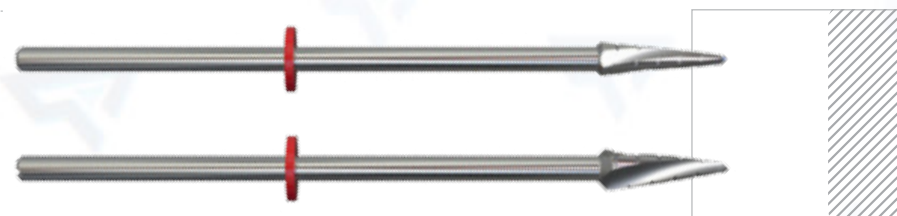

Cylindrical burrs
for conical cutting
& extraction

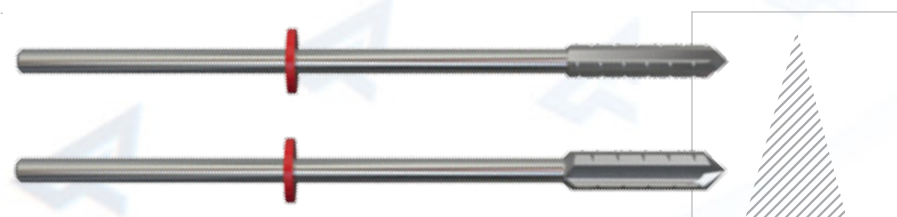STEP 5 - PROTECTION SLEEVE

TECHNICAL FEATURES

- *Protection:* prevents soft tissue irritation.
- *Adaptability:* 2 diameters (3.1 mm & 4.1 mm) available.
- *Compatibility:* adapts to main micromotors.

CURETTES

USE

- *To remove small bones debris.*
- *To pass under the periost and create a "protection room".*
- *To feel direct contact with the bone.*
- *To unstick the capsula.*

TECHNICAL FEATURES

- *Ergonomy:* anatomic silicone handle.
- *Versatility:* many extremities shapes.
- *Specificity:* mini-invasive surgery teeth design.
- *2 sizes available:* medium and large.

INSTRUMENTS AND TOOLS

1000319 DPR system instrument tray, w/o instruments

1201155 Burr straight
ø 2.0 x L. 15 mm

1201156 Burr conical
ø 3.1/1.0 x L. 12 mm

1201157 Burr straight
ø 1.9 x L. 10 mm

1201158 Burr conical
ø 4.1/1.0 x L. 12 mm

1201159 Burr cylindrical
ø 4.1 L. 15 mm

1201160 Burr cylindrical
ø 3.1 L. 15 mm

1028114 Curette medium - down

1028115 Curette medium - up

1028116 Curette large - down

1028117 Curette large - up

1028118 Sleeve protection cap

1028119 Sleeve protection dia. 4.1 mm

1028120 Sleeve protection dia. 3.1 mm

MORE OF OUR PRODUCTS


2010002

Foot & Ankle
system


2010015

Cannulated
Screws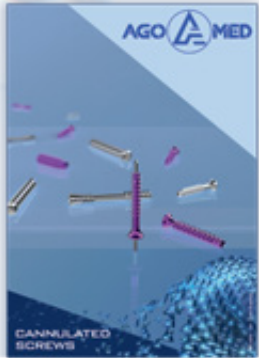

2010003

Titanium plating system
for craniomaxillofacial osteosynthesis

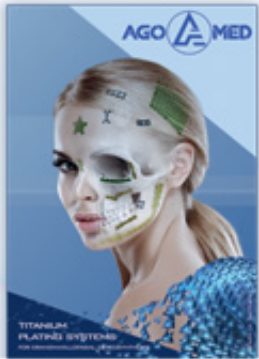2010012

Dragon
Foot & Ankle system


2010004

Plating system for
Upper Extremities


2010008

AgoFix Neuro
Plating system 1.5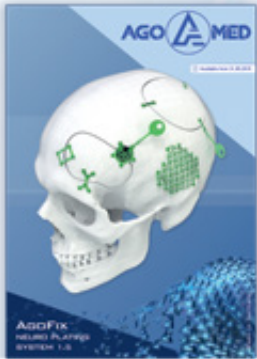


2010017

AGO-Pediatric
system 3.5 / 4.0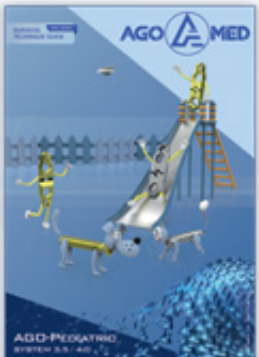

2010007

King Kong Multidirectional
Radius system 2.5

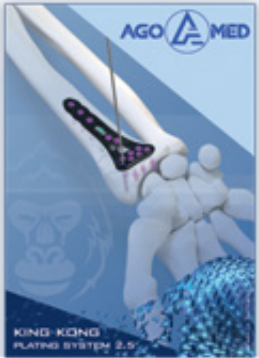2010016

Titanium Mesh systems for
craniomaxillofacial osteosynthesis / Neuro

NOTES

Blank area for notes.

NOTES

Handwriting practice area with horizontal lines and a background pattern of stylized 'A' shapes.

NOTES

Handwriting practice area with horizontal lines and a background pattern of stylized 'A' shapes.

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